IALU Fresh®

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Zone Industrielle Kalâat El Andalous - 2022 Ariana - TUNISIE

IALU**fresh**[®]

- Please read carefully the entire instructions of this leaflet: it contains important information for the user.
- Keep this leaflet; you may need to read it again.

IALU**fresh**[®]

Hyaluronic acid

Form and presentation:

IALU**fresh**[®] buccal gel is presented out of 20 g bottle with cannula as end piece.

IALU**fresh**[®] buccal spray is presented out of 30 mL bottle with multidirectional spray.

Composition:

IALU**fresh**[®] **buccal gel:** Hyaluronic acid (0.24 %), purified water, xylitol, propylene glycol, hydrogenated castor oil PEG 60, polyvinyl alcohol, carmellose sodium, 2,4-dichlorobenzyl alcohol, liquid mint flavour, carbomers, Sodium hydroxide, Patent blue.

IALU**fresh**[®] **buccal spray:** Hyaluronic acid (0.1%), purified water, propylene glycol, xylitol, hydrogenated castor oil PEG 60, polyvinyl alcohol, 2,4- dichlorobenzyl alcohol, liquid mint flavor, sodium hydroxide.

Properties:

Hyaluronic acid is a natural component of the connective tissue especially in the gingival mucosa where it helps to facilitate the natural tissue healing and recovery process.

IALU**fresh**[®] is a medical device (Class IIa) containing hyaluronic acid (as a salt) of high molecular weight.

Its formulation allows it to:

- Form an effective mechanical barrier to mouth sores, soothing the pain immediately.
- Promote hydration and regeneration of injured tissues.
- Assume a maximum adherence to injured lesions, for efficient, continuous and sustainable action.

IALU**fresh**[®] contains xylitol, a non-cariogenic natural sweetener and does not induce significant changes in blood glucose levels.

The product is also suitable for diabetics. It has no drug interaction.

Indications:

IALU**fresh**[®] is intended for:

- Inflammatory states: gingivitis, bleeding, gingival recession, gingival pockets.
- Traumatic states: canker sores, minor wounds, lesions due to wearing prosthetic mouth, abrasion, dental extractions, post-surgical sequelae.

Mode and duration of use:

IALU**fresh**[®] **buccal gel:** Apply 2 to 3 times a day, or more if needed, for a week or until the symptoms disappear.


If the lesion is difficult to reach, apply the product with a clean finger.

It is recommended to reverse the bottle and to press the body to deliver the product. Within 2 minutes of application, it is recommended to avoid contact between the tongue and the lesion to allow the film to form.

IALU**fresh**[®] **buccal spray:** Apply 1 or 2 sprays on the inflamed or traumatized tissues, 3 to 5 times a day for a week or until the symptoms disappear.

To spray, just rotate the nozzle tip a quarter of a turn. Then, orient the nozzle towards the lesion and spray.

SPRAY BY KEEPING THE BOTTLE UPRIGHT

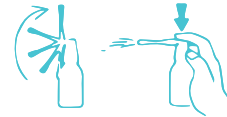Use IALU**fresh**[®] after eating and brushing teeth.

In case of persistent pain, the product can be safely applied several times per day. It is disadvised to eat or drink for 30 minutes after application.

Safe if ingested.

In the case of a person who is not self-sufficient, recourse to the assistance of another person is possible.

Contraindications:

- Do not use IALU**fresh**[®] in case of known hypersensitivity to any of its constituents. If local hypersensitivity reactions appear and persist, consult your doctor.
- If symptoms persist, consult your doctor.
- Do not use for children under 3 years old.

Precautions for use / Warnings:

- Keep out of reach of children.
- Avoid any direct contact with the device's applicator.
- Close the product immediately after use.
- The expiry date relates to a closed and stored under proper conditions.
- Do not use the product if it is not intact and / or damaged.
- IALU**fresh**[®] is for strictly personal use.
- Avoid contact of the spray with eyes.
- Do not dispose of the medical device in the sewage system or with household waste. Ask your pharmacist what to do with medical devices that are no longer used. These measures will help protect the environment.

Storage conditions:

Keep the medical device away from heat, preferably at a temperature not exceeding 25 ° C.

Leaflet revision date:

June 2021.

LOT

Opalia Pharma S.A., Recordati group
Z.I. Kalâat El Andalou 2022 Ariana - Tunisie - Tunisie
www.opaliapharma.com

MD

opalia
Pharma

RECORDATI GROUP

OPALIA PHARMA (RECORDATI GROUP)

Zone Industrielle Kalâat El Andalou - 2022 Ariana - TUNISIE